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*Foto-manual: monitoring kit for reuseable transducer SP 844 and SP 840*

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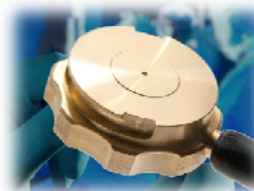
*A monitoring kit consists in general:*

*infusion tubing with roller clamp*

*non-vented drip chamber*

*# Click dome with flush-device and stopcock, if using the transducer SP844*

*# Bayonet dome with flush-device and stopcock, if using the transducer SP840*

*pressure line, which can be transparent or red resp. blue*

*3-way stopcock and if applicable a patient extension*

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*Find successive explanation, how to use the reuseable transducer SP844 or SP840.*

Nach Montage von Stativklemme und Transducer-Halter den Transducer befestigen  
*After mounting of tripod bracket and transducer holder, fasten transducer*

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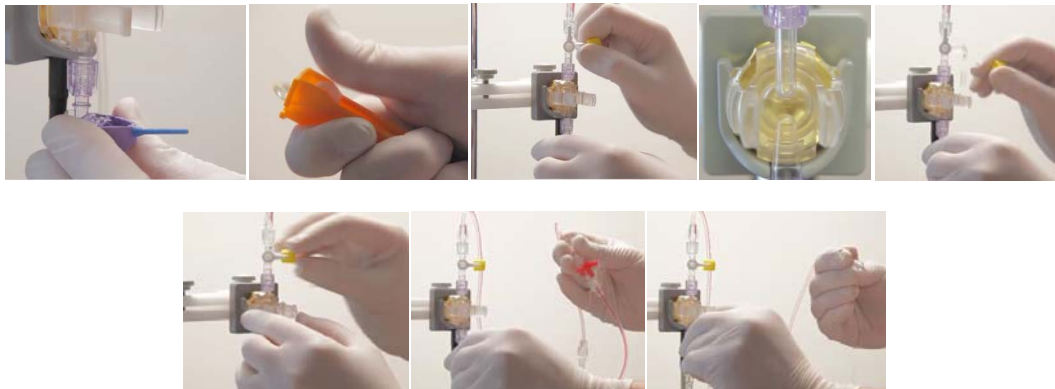
*After use, discard the Monitoring Set and clean the transducer as necessary (with a moist, sterile cloth), place a protective cap (or a single dome) on the transducer and store it in a safe place.*

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**Please observe:** Invasive blood pressure measurements should be carried out by or under the supervision of qualified personnel, only. The finally wished configuration of the monitoring kit and the procedure of invasive pressure measuring is varying according the basic orientations of particular operator. Responsible of defining the exact regulations and procedures is the operator respectively the Hospital. To obtain perfect pressure measurements with the SP844 / SP840 and the monitoring kit and secure the safety of the patient, the instruction in the manual should be observed and carried out in the sequence stated.

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